





## Favorite User Key Options

**Tabular Trend:** This key displays a chart of all user defined patient vital signs.

- Touch **Tabular Trend**
  - ◆ Select **Interval** to change the frequency of the vital sign display from 10 sec to 60 min.
  - ◆ To Print - Touch **Print** on the bottom right.

**Graphic Trend:** This section displays user defined graphs and trends of various vital signs and events.

- **Menu** → **Function** → **Graphic Trend** if not found as a user key.

**Recall:** This button will allow you to view the alarmed events on the patient.

- Touch **Recall** > Touch the "**Alarmed Event**" you wish to view and a 4 second strip will be displayed.
  - ◆ To print the alarm, touch **Print** in the bottom right corner.

**Other Bed:** This key displays waveforms, vital signs and events of a second patient on the screen.

- Touch **Other Bed** located next to the patient data at the top of the screen > Select the bed you want to view > Size and a picture in picture will appear with the second patient's data.

**\*Note: This function only works on a hardware installation.**

**Night Mode:** This will adjust the brightness, turning off the alarm sounds at the bedside while alarms and vital signs remain audible and seen at the Central Station.

- Touch the **Night Mode** User Key. To cancel, touch the **Power Quit** key on the screen.

**Display Config:** This option allows you to add or subtract numerics or waveforms from the current display.

- **Menu** → **Display Config** → **Choose Configuration** → **Waveform** → **Numerics**

### Labeling & Zeroing (Invasive)

- ◆ Change the mode by touching the **Admit/Discharge** key or the **Mode** found at the top of the screen.
- ◆ Touch the BP numeric and to open the menu.
- ◆ Touch **Label** and a menu with all of the available labels will be displayed. Choose the appropriate Label. The color of the waveform, the scale and the numerics will change according to the Label.
- ◆ With the stop cock open to air, Touch and hold **BP Zero for 3 seconds**. The zeroing is complete when you hear the beep and see the message "**BP zero complete**". Close stop cock to air to begin monitoring.

