



DS-8400

Mounting Solution Guide

Configuration

DS-8400 Bedside Monitor Mounting

DS-8007 - Portable
Bedside Monitor

DSC-8410 -Main Unit




LC-8016TC - 15.6" Display Unit
LC-8018TC - 18.5" Display Unit

Dock on the Back



OR

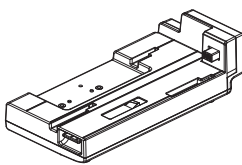

DS-8007M

DS-8007N

Masimo® Model Nellcor™ Model

HSA-81

HS Adapter

HS-8312M


Masimo® HS Module Nellcor™ HS Module

HS-8312N

Separate Connection

Module Connection Cable

*Connect to Main Unit DSC-8410

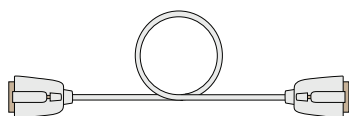

○ **CJO-08SS1.5** (1.5M)

Special Order Item

- **CJO-08SS0.3** (0.3M)
- **CJO-08SS3.5** (3.5M)
- **CJO-08SS5** (5M)




DSA-82
DS-8007 Adapter



DS-8007M
Masimo® Model

DS-8007N
Nellcor™ Model

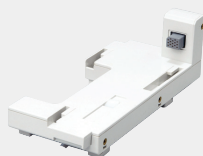

HSA-80
HS- Adapter



HS-8312M
Masimo® HS Module

HS-8312N
Nellcor™ HS Module

***Can fit one module in the DSC-8410 unit - requires input box for more than one**



IB-8004
Input Box



HG-820
Nellcor™
SpO2 Module



HG-810
Masimo®
SpO2 Module



HM-800
Multi-Module




HP-800
Multi Port
Module




HM-801
Multi-Module

DS-8400 with HS-8312N/M Docked on the Back Configuration

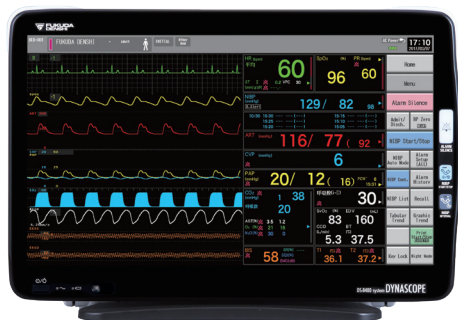


DSC-8410
Main CPU
+
LC-8016TC/LC-8018TC
Display Unit

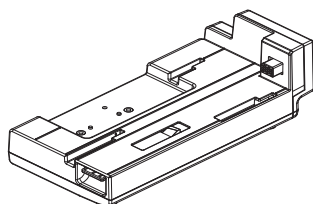

HSA-81
HS Adapter



HS-8312N/M
HS Super Module

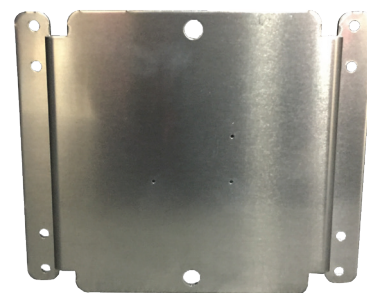

OAO-70A
GCX® Attachment
for Monitor

Mounting Parts



WMM-0006-04
Dual Utility Hook

OR



WS-0003-26A
Quad Utility Hook

Arm Options




FD-0009-21D
Wall Mount
Fixed Arm

OR



FD-0033-01
Wall Mount
Variable Height


DS-8400 with HS-8312N/M with EtCO2 or BISx Docked on the Back Configuration

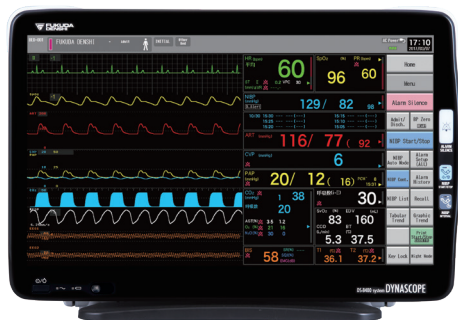

DSC-8410
Main CPU

+

LC-8016TC/LC-8018TC
Display Unit

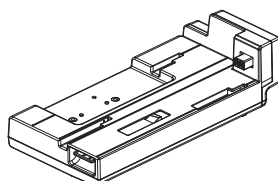

HSA-81
HS Adapter



HS-8312N/M
HS Super Module



CJO-15RR0.65G
AUX Connection Cable
*Use with OAO-100A Mounting Bracket




HBX-800
BISx Module Interface

OR



HCP-810/HPD-810
EtCO2 Unit

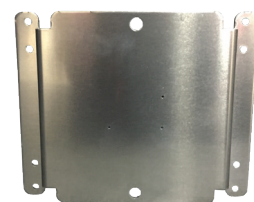

OAO-70A
GCX® Attachment
for Monitor

Mounting Parts




WMM-0006-04
Dual Utility Hook

OR



WS-0003-26A
Quad Utility Hook

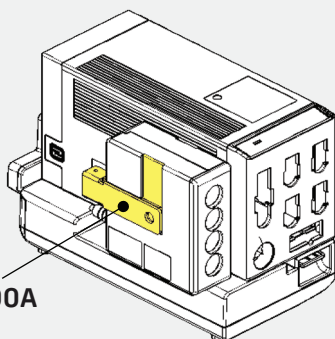

OAO-100A

OAO-100A
Module
Mounting Bracket

Arm Options



FD-0009-21D
Wall Mount
Fixed Arm


OR




FD-0033-01
Wall Mount
Variable Height

DS-8400 with EtCO2 Module (GCX® Mounting)

Configuration

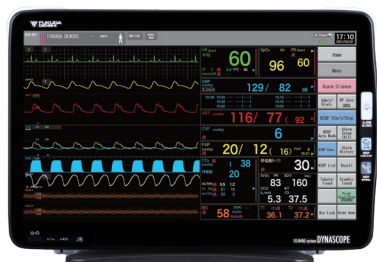

DSC-8410
Main CPU
+
LC-8016TC/LC-8018TC
Display Unit

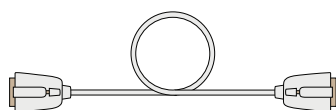

CJO-08SS1.5
Module Connection Cable



CJO-15RR0.65G
AUX Connection Cable
*Use with OAO-59A Clamp Holder



HS-8312N/M
HS Super Module
with **HSA-80** HS Adapter



HCP-810
Sidestream Unit



HPD-810
Mainstream Unit

Mounting Parts



WMM-0006-04
Dual Utility Hook

OR



WS-0003-26A
Quad Utility Hook




FD-0030-80
Mounting Tray
for HS-8312N/M



FD-0009-21D
Wall Mount
Fixed Arm

OR

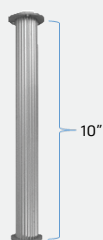

WMM-0006-11
Downpost (10")

*Single Stage Fixing
Post for Modules



FD-0027-83
Single-sided
Downpost-tray
Adapter



OAO-59A
HPD/HCP
Clamp Holder
*Can be used without the clamp




FD-0033-01
Wall Mount
Variable Height

DS-8400 with Recorder & EtCO2 Module (GCX® Mounting) Configuration




DSC-8410
Main CPU
+
LC-8016TC/LC-8018TC
Display Unit

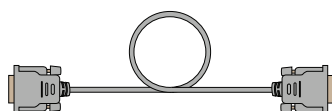

CJO-09SS1.5
Unit Connection Cable
(U-LINK)



HR-800
3CH Thermal Printer

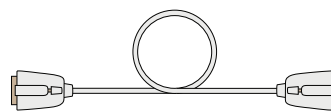

CJO-08SS1.5
Module Connection Cable



HS-8312N/M
HS Super Module
with **HSA-80** HS Adapter



CJO-15RR0.65G
AUX Connection Cable
*Use with OAO-59A Clamp Holder



HCP-810
Sidestream Unit

OR



HPD-810
Mainstream Unit

Mounting Parts




WMM-0006-04
Dual Utility
Hook

OR



WS-0003-26A
Quad Utility Hook

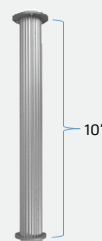

WMM-0006-11
Downpost (10'')
*Single Stage Fixing
Post for Modules



FD-0009-21D
Wall Mount
Fixed Arm

OR



FD-0028-80
Mounting Tray
for HR-800



FD-0030-80
Mounting Tray
for HS-8312N/M



FD-0027-84
Double-sided
Downpost-tray
Adapter



OAO-59A
HPD/HCP
Clamp Holder
*Can be used without
the clamp




FD-0033-01
Wall Mount
Variable Height

DS-8400 with IB-8004 Input Box Unit, Recorder & EtCO2 Module (GCX® Mounting) Configuration




DSC-8410
Main CPU
+
LC-8016TC/LC-8018TC
Display Unit

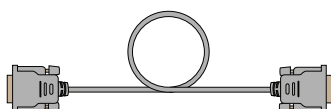

CJO-09SS1.5
Unit Connection Cable
(U-LINK)



HR-800
3CH Thermal Printer

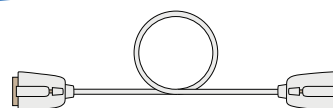

CJO-08SS1.5
Module Connection Cable



IB-8004
Input Box



HS-8312N/M
HS Super Module
with **HSA-80** HS Adapter



CJO-15RR0.65G
AUX Connection Cable
*Use with OAO-59A Clamp Holder



HCP-810
Sidestream Unit

OR



HPD-810
Mainstream Unit

Mounting Parts



WMM-0006-12
Downpost (16")
*Dual Stage Fixing
Post for Modules



FD-0028-80
Mounting Tray
for HR-800



FD-0030-80
Mounting Tray
for HS-8312N/M



FD-0031-80
Mounting Tray
for IB-8004



FD-0009-21D
Wall Mount
Fixed Arm

OR



FD-0027-83
Single-sided
Downpost-tray
Adapter



FD-0027-84
Double-sided
Downpost-tray
Adapter



OAO-59A
HPD/HCP
Clamp Holder
*Can be used without
the clamp



WMM-0006-04
Dual Utility Hook

OR



WS-0003-26A
Quad Utility Hook




FD-0033-01
Wall Mount
Variable Height


DS-8400 with BISx Module (GCX® Mounting) Configuration



DSC-8410
Main CPU
+
LC-8016TC/LC-8018TC
Display Unit

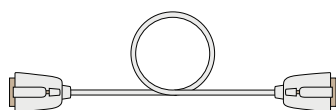

CJO-08SS1.5
Module Connection Cable

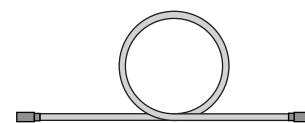

CJO-15RR0.65G
AUX Connection Cable
*Use with OAO-59A Clamp Holder



HS-8312N/M
HS Super Module
with **HSA-80** HS Adapter



HBX-800
BISx Module Interface

Mounting Parts



WMM-0006-04
Dual Utility Hook

OR



WS-0003-26A
Quad Utility Hook




FD-0030-80
Mounting Tray
for HS-8312N/M



FD-0009-21D
Wall Mount
Fixed Arm

OR

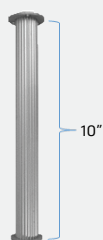

WMM-0006-11
Downpost (10")

*Single Stage Fixing
Post for Modules



FD-0027-83
Single-sided
Downpost-tray
Adapter



OAO-59A
HPD/HCP
Clamp Holder
*Can be used without the clamp




FD-0033-01
Wall Mount
Variable Height

DS-8400 with Recorder & BISx Module (GCX® Mounting) Configuration




DSC-8410
Main CPU
+
LC-8016TC/LC-8018TC
Display Unit

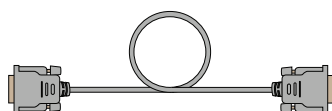

CJO-09SS1.5
Unit Connection Cable
(U-LINK)



HR-800
3CH Thermal Printer

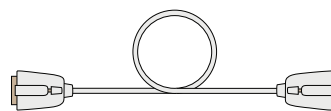

CJO-08SS1.5
Module Connection Cable



HS-8312N/M
HS Super Module
with **HSA-80** HS Adapter



CJO-15RR0.65G
AUX Connection Cable
*Use with OAO-59A Clamp Holder



HBX-800
BISx Module Interface

Mounting Parts




WMM-0006-04
Dual Utility
Hook

OR



WS-0003-26A
Quad Utility Hook

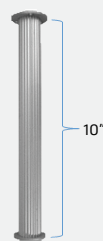

WMM-0006-11
Downpost (10")
*Single Stage Fixing
Post for Modules



FD-0009-21D
Wall Mount
Fixed Arm

OR



FD-0028-80
Mounting Tray
for HR-800



FD-0030-80
Mounting Tray
for HS-8312N/M



FD-0027-84
Double-sided
Downpost-tray
Adapter




OAO-59A
HPD/HCP
Clamp Holder
*Can be used without
the clamp




FD-0033-01
Wall Mount
Variable Height

DS-8400 with IB-8004 Input Box Unit, Recorder & BISx Module (GCX® Mounting) Configuration

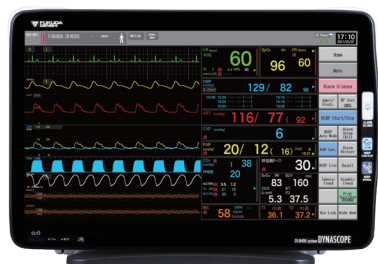


DSC-8410
Main CPU
+
LC-8016TC/LC-8018TC
Display Unit

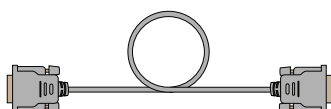

CJO-09SS1.5
Unit Connection Cable
(U-LINK)



HR-800
3CH Thermal Printer

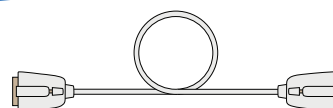

CJO-08SS1.5
Module Connection Cable



IB-8004
Input Box



HS-8312N/M
HS Super Module
with **HSA-80** HS Adapter



CJO-15RR0.65G
AUX Connection Cable
*Use with OAO-59A Clamp Holder



HBX-800
BISx Module Interface

Mounting Parts



WMM-0006-12
Downpost (16")
*Dual Stage Fixing
Post for Modules



FD-0028-80
Mounting Tray
for HR-800



FD-0030-80
Mounting Tray
for HS-8312N/M



FD-0031-80
Mounting Tray
for IB-8004



FD-0009-21D
Wall Mount
Fixed Arm

OR



FD-0027-83
Single-sided
Downpost-tray
Adapter



FD-0027-84
Double-sided
Downpost-tray
Adapter



OAO-59A
HPD/HCP
Clamp Holder
*Can be used without
the clamp



WMM-0006-04
Dual Utility Hook

OR



WS-0003-26A
Quad Utility Hook



FD-0033-01
Wall Mount
Variable Height

DS-8400 with MGU-801 Gas Module and Recorder (GCX® Mounting) Configuration




DSC-8410

Main CPU

+


LC-8016TC/LC-8018TC

Display Unit

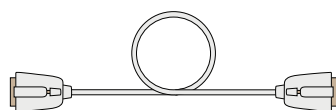

CJO-08SS1.5

Module Connection Cable

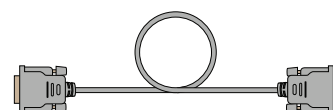

CJO-09SS1.5

Unit Connection Cable (U-LINK)



HS-8312N/M

HS Super Module
with **HSA-80** HS Adapter



MGU-801P

AGO2 Gas Unit



HR-800

3CH Thermal Printer

Mounting Parts



WMM-0006-04

Dual Utility
Hook

OR



WS-0003-26A

Quad Utility
Hook



WMM-0006-12
Downpost (16")

*Dual Stage Fixing
Post for Modules



FD-0028-80
Mounting Tray
for HR-800



FD-0009-21D

Wall Mount
Fixed Arm

OR



FD-0030-80
Mounting Tray
for HS-8312N/M



FD-0032-80
Mounting Tray
for MGU-801P



FD-0027-83
Single-sided
Downpost-tray
Adapter




FD-0027-84
Double-sided
Downpost-tray
Adapter




FD-0033-01
Wall Mount
Variable Height

DS-8400 with All Modules and Units (GCX® Mounting)


Configuration

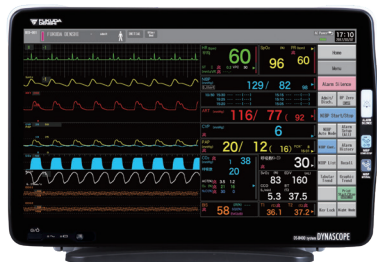

DSC-8410
Main CPU
+
LC-8016TC/LC-8018TC
Display Unit

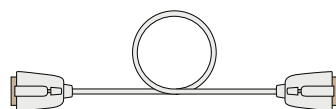

CJO-08SS1.5
Module Connection Cable

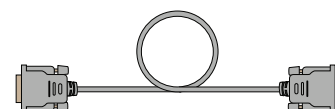

CJO-09SS1.5
Unit Connection Cable (U-LINK)



HS-8312N/M
HS Super Module
with **HSA-80**
HS Adapter



IB-8004
Input Box



MGU-801P
AGO2 Gas Unit



HR-800
3CH Thermal Printer

Mounting Parts



WMM-0006-04
Dual Utility
Hook

OR



WS-0003-26A
Quad Utility
Hook



WMM-0006-12
Downpost (16")
*Dual Stage Fixing
Post for Modules



FD-0028-80
Mounting Tray
for HR-800



FD-0009-21D
Wall Mount
Fixed Arm

OR



FD-0030-80
Mounting Tray
for HS-8312N/M



FD-0031-80
Mounting Tray
for IB-8004



FD-0032-80
Mounting Tray
for MGU-801P




FD-0027-84 x2
Double-sided
Downpost-tray
Adapter



FD-0033-01
Wall Mount
Variable Height

DS-8400 Module Configuration for Fukuda Mounting

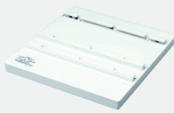 HS-8312 Super Module	 HCP-810/HPD-810 EtCO2	 HBX-800 BISx	 HR-800 Recorder	 MGU-801P Gas Unit	 IB-8004 Input Box	 OAO-52A HS Fixing Base	 MXD-M3-36617-01 Dual Base Mounting Tray
●	●					1	
●		●				1	
●			●			1	
●				●		1	
●					●	1	
●	●		●			2	1
●		●		●		2	1
●			●	●		2	1
●	●		●		●	3	1
●		●		●	●	3	1
●			●	●	●	3	1


DS-8400 with Up to 2 Modules (Fukuda Mounting)

MONITOR CONFIGURATION



DSC-8410
Main CPU

LC-8016TC
+ **LC-8018TC**
Display Unit

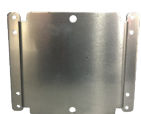

OAO-70A
GCX® Attachment
for Monitor

MODULE CONFIGURATION



HS-8312N/M HS Super Module
with **HSA-80** HS Adapter

PICK ONE OTHER MODULE



HCP-810/HPD-810
EtCO2 Unit



HBX-800
BISx Module
Interface



HR-800
3CH Thermal
Printer



MGU-801P
AGO2
Gas Unit



IB-8004
Input Box



with **OAO-59A**
Holder



with **OAO-59A**
Holder

Arm Options




FD-0009-21D
Wall Mount
Fixed Arm

OR




FD-0033-01
Wall Mount
Variable Height

Mounting Parts

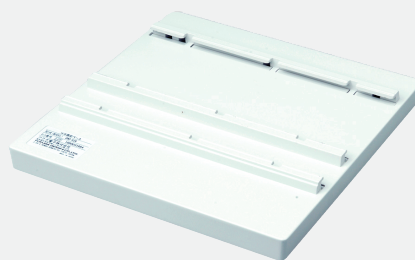

OAO-52A
HS Fixing Base

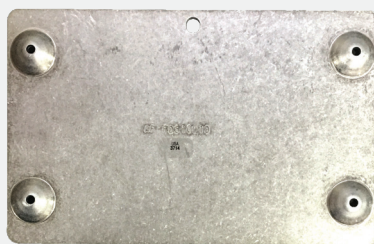

FD-0020-16
Base Plate Adapter


Arm Option




WMM-0012-06
M Series Fixed Height
Arms (8 inch)

DS-8400 with Up to 3 Modules (Fukuda Mounting)

MONITOR CONFIGURATION

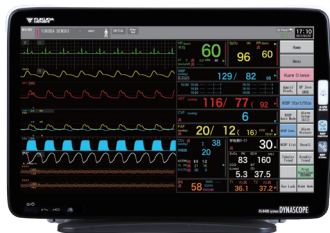

DSC-8410 + **LC-8016TC**
Main CPU + LC-8018TC
Display Unit

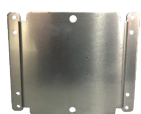

OAO-70A
GCX® Attachment
for Monitor

MODULE CONFIGURATION



HS-8312N/M HS Super Module
with **HSA-80** HS Adapter

PICK TWO OTHER MODULE



HCP-810/HPD-810
EtCO2 Unit



HBX-800
BISx Module
Interface



HR-800
3CH Thermal
Printer



MGU-801P
AGO2
Gas Unit



IB-8004
Input Box

OR

OR

OR

OR



with **OAO-59A**
Holder



with **OAO-59A**
Holder

Arm Options




FD-0009-21D
Wall Mount
Fixed Arm

OR




FD-0033-01
Wall Mount
Variable Height

Mounting Parts

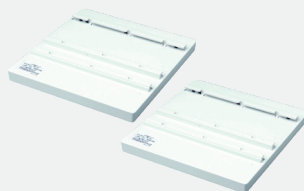

OAO-52A x 2
HS Fixing Base

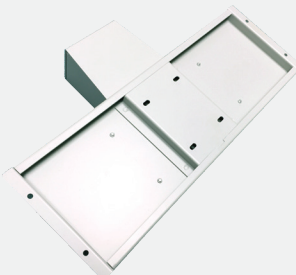

MXD-M3-36617-01
Dual Base Mounting
Tray




FD-0020-16
Base Plate Adapter

Arm Option



WMM-0012-06
M Series Fixed Height
Arms (8 inch)


DS-8400 with DS-8007N/M Docked on the Back Configuration

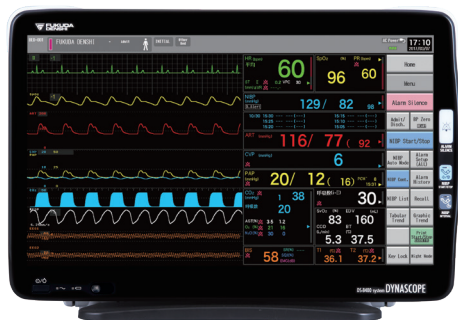

DSC-8410
Main CPU
+
LC-8016TC/LC-8018TC
Display Unit



DS-8007M **DS-8007N**
Masimo® Model Nellcor™ Model

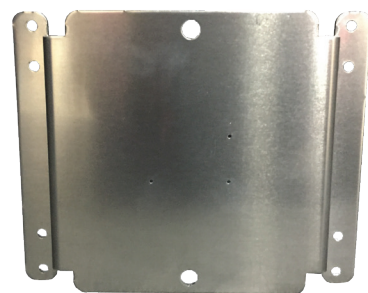

OAO-70A
GCX® Attachment
for Monitor

Mounting Parts



WMM-0006-04
Dual Utility Hook

OR



WS-0003-26A
Quad Utility Hook

Arm Options



FD-0009-21D
Wall Mount
Fixed Arm

OR



FD-0033-01
Wall Mount
Variable Height

DS-8400 with DS-8007N/M with EtCO2 Docked on the Back Configuration



DSC-8410
Main CPU
+
LC-8016TC/LC-8018TC
Display Unit



DS-8007M
Masimo® Model



DS-8007N
Nellcor™ Model



CJO-25TR-0.65
AUX LEM Connection Cable
*Use with OAO-103A Mounting Bracket





HCP-810/HPD-810
EtCO2 Unit

OR



HCP-820/HPD-820
EtCO2 Unit

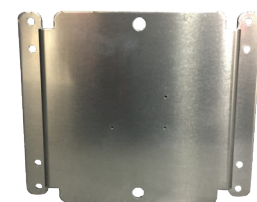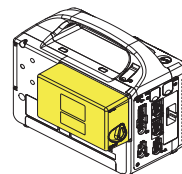

OAO-70A
GCX® Attachment
for Monitor

Mounting Parts




WMM-0006-04
Dual Utility Hook

OR



WS-0003-26A
Quad Utility Hook

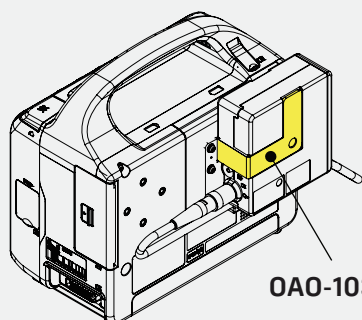

OAO-103A

OAO-103A
AUX Unit
Mounting Bracket

*Do not need if using HCP-820/HPD-820 EtCO2 Unit

Arm Options



FD-0009-21D
Wall Mount
Fixed Arm

OR



FD-0033-01
Wall Mount
Variable Height

DS-8400 with DS-8007N/M with BISx Docked on the Back Configuration




DSC-8410
Main CPU
+
LC-8016TC/LC-8018TC
Display Unit



DS-8007M
Masimo® Model
DS-8007N
Nellcor™ Model



CJO-25TR-0.65
AUX LEM Connection Cable
*Use with OAO-103A Mounting Bracket

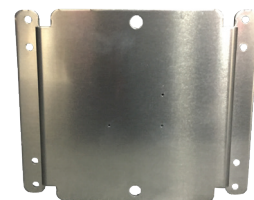

OAO-70A
GCX® Attachment
for Monitor



HBX-800
BISx Module Interface

Mounting Parts




WMM-0006-04
Dual Utility Hook

OR



WS-0003-26A
Quad Utility Hook

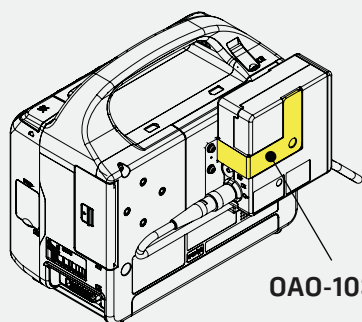

OAO-103A

OAO-103A
AUX Unit
Mounting Bracket

Arm Options



FD-0009-21D
Wall Mount
Fixed Arm

OR



FD-0033-01
Wall Mount
Variable Height


DS-8400 with DS-8007 Connected Separately

MONITOR CONFIGURATION




DSC-8410
Main CPU

LC-8016TC
+ **LC-8018TC**
Display Unit

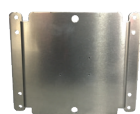

OAO-70A
GCX® Attachment
for Monitor

DS-8007 CONFIGURATION

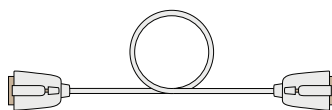

CJO-08SS1.5
Module Connection Cable


*Connect to Main Unit DSC-8410



DS-8007N/M
Main Unit



DSA-82
DS-8007 Adapter

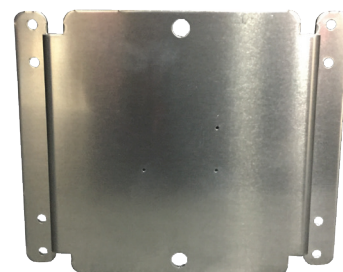

OAO-70A
GCX® Attachment
for Monitor

Arm Options



FD-0009-21D
Wall Mount
Fixed Arm

OR



FD-0033-01
Wall Mount
Variable Height

Mounting Parts



WMM-0006-04
Dual Utility Hook

OR



WS-0003-26A
Dual Utility Hook

Arm Options



FD-0009-21D
Wall Mount
Fixed Arm

OR



FD-0033-01
Wall Mount
Variable Height



FUKUDA DENSHI USA, INC.
17725-C NE 65th Street Redmond, WA 98052
Toll Free: (800) 365-6668 / Local: (425) 881-7737 / Fax: (425) 869-2018
www.fukudaamerica.com

GCX® is a registered trademark of GCX Corporation.
Nellcor™ is a registered trademark of Medtronic.
Masimo® is a registered trademark of Masimo Corporation.
The information contained in this document is subject to change
without notice. © 2022 FUKUDA DENSHI USA, INC. All rights reserved.