



4 & 5 Lead ECG Placement Reference Guide

*Recommend **ECG electrodes** be changed every 24 hours.*

Green Lead: place anywhere on the lower Right area of chest

White & Black Lead: placed mid clavicular (below the clavicle bone).

White on the Right

Black on the Left

***Note:** If the patient has a pacemaker, place black lead on the trapezius muscle (top fatty portion of the shoulder).

Red Lead: placed between 5th and 8th intercostal space along the mid-axillary line

Brown Lead: place at the 4th intercostal space to the Right (as shown) or to the Left of the sternal boarder.

