

## FX-8400 Specifications

## ECG

Sensitivity Selection	1/4, 1/2, 1, 2, Auto
Differential and Common-Mode Offset Voltage (Electrode-Skin Voltage)	±600 mV and above
Sine Wave Characteristics	0.05 Hz to 250 Hz
Low Frequency Characteristics (Time Constant)	3.2 sec. and above
Common-Mode Signal Suppression	103 dB and above (2 mm [p-v] and below at sensitivity level 1)
Leads	Standard 12-lead
Filter	AC filter: -20 dB or less at 50 Hz or 60 Hz Muscle filter: -3 dB (-6 dB/oct) or less at 35 Hz or 25 Hz Drift: -3 dB (-12 dB/oct) or less at 0.25 Hz or 0.5 Hz
Printing	Thermal Print Head Method
Printing Speed	5, 10, 12.5, 25, 50 mm/s ±2% or less
Printing Channel	3 ch, 6 ch, 12 ch,
Printing Paper	(Roll paper with grids, 210mm) OP-69TE (Z-fold paper with grids, 210mm) OP-618TE, OP-621TE
Display	8" Color LCD, 800 × 480 dots (with LED backlight)
A/D Conversion	24-bit
Sampling Rate	8,000 samples/sec.
LAN Port	Conforms to IEEE802.3u 100BASE-TX (The cable must be within 50 m.)
USB Port	Compatible with USB2.0 Full Speed, 3 ports
SD Card	Compatible with SD Card Specification 2.0
Serial Port	Compatible with RS-232C, 2 ports

## Equipment

Power Supply	AC power: AC 100-240V 50/60 Hz DC power :14.8V DC (Battery)
Power Consumption	100 VA (AC)
Dimensions (W x D x H)	Approx. 330 × 350 × 85 mm / 13.0 x 13.8 x 3.3 inch (not including the protrusion) Approx. 330 × 350 × 112.6 mm / 13.0 x 13.8 x 4.4 inch (including the protrusion)
Weight	Approximately 4.0 kg / 8.8 lbs (main unit only) Approximately 4.5 kg / 9.9 lbs (including options such as battery)
Battery operation time	240min

## System Components

Li-ion Battery Pack  
(BTE-002)ECG INTERPRETATION SOFTWARE  
(FP-811)

## Multiple Languages

English, French, German, Spanish, Italian, Russian, Portuguese, and Vietnamese

**FUKUDA** | Accessible Healthcare for Everyone

FUKUDA DENSHI USA, INC.  
17725-C NE 65th Street Redmond, WA 98052  
Toll Free: (800) 365-6668 / Local: (425) 881-7737 / Fax: (888) 224-7090

The information contained in this document is subject to change without notice.  
© 2023 FUKUDA DENSHI USA, INC. All rights reserved. Document No. USRB-0270-00

<http://www.fukudaamerica.com>


# Electrocardiograph

## FX-8400

# CardiMax

# FX-8400

## Electrocardiograph


**FUKUDA**  
Accessible Healthcare for Everyone

The FX-8400 has a large 8-inch widescreen LCD offering unparalleled ease of use for checking patient data, as well as clear, accurate ECG waveforms.

### 8-inch Widescreen Color LCD

A large 8-inch color display with clear waveforms and easy to operate.

### Bar Code/ID Card Reader (Optional)

Enables the user to enter patient information with the bar code reader or ID card reader for quick and error-free input.



### Custom Key

The user can register up to four (4) customizable, shortcut keys for functions such as cueing the input of patient information, toggling filter on or off, manual recording, and auto-printing.

### Memory Function

Up to 1,000 ECG examinations can be saved in the cardiograph's internal memory. Examination data can also be saved on the removable SD card or USB stick (optional).



### Wireless LAN (Optional)

By connecting the optional wireless LAN adapter, communication with the data management system is possible without having to connect cables.



### Function Key

By setting up the function keys, the user can create short-cuts to frequently used functions. This enables more efficiency and flexibility during operation.

### Convenient Paper Tray

Both Z-fold and roll paper can be set inside the FX-8400.



### Full QWERTY Keyboard

A full QWERTY keyboard, standardly provided, facilitates efficient input of patient information and other test information. This keyboard may be ordered in seven (7) other language formats.





# Enhanced Functionality for More Electrocardiography (Included in Optional FP-811)

## Superior Detection of Acute Coronary Syndrome (ACS)

### ACS Diagnostic Support Function

For more effective identification of acute myocardial infarction, the FX-8400 utilizes an 18-lead ECG test (which includes some synthesized leads) and a specialized ACS diagnosis algorithm.



#### Synthesized 18-lead

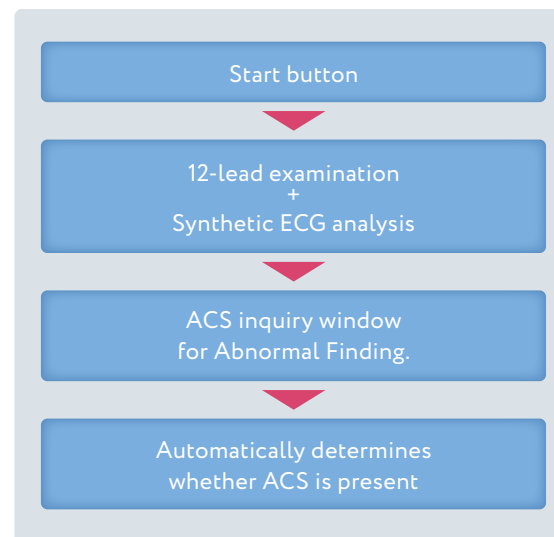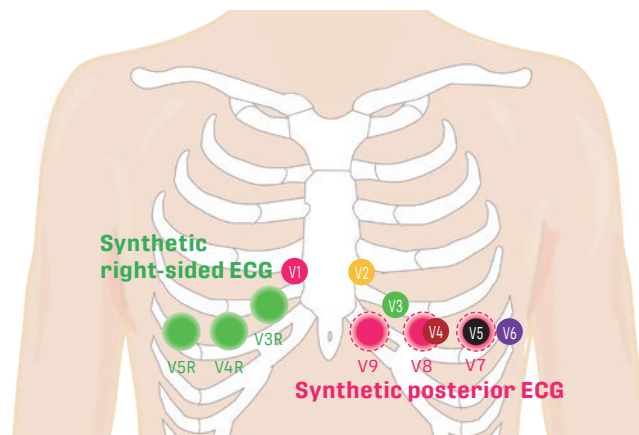

The additional waveforms are calculated using examination data obtained from a 12-lead ECG (additional electrodes are not required).

The ST levels of the right side leads (V3R, V4R, and V5R) and posterior wall leads (V7, V8, and V9) are analyzed to diagnose ACS.

#### Analysis Screen

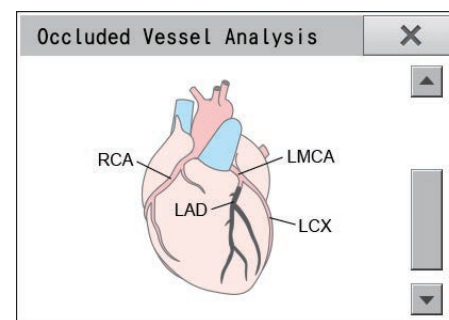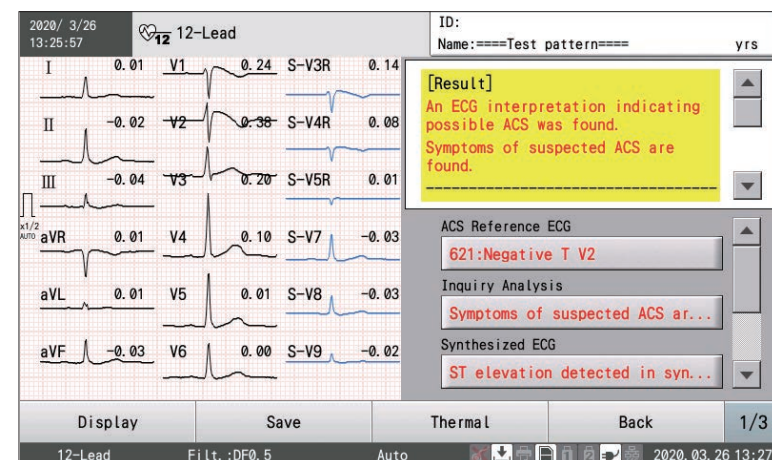


The FX-8400 will produce an interpretation and commentary relevant to ACS, as well as an analysis, shown graphically, that helps to identify the location of the occluded coronary vessel(s).

## Predicting and Analyzing ECG Abnormalities Associated with Sudden Cardiac Death

### Brugada Risk Analysis


Brugada syndrome is normally detected when placing the leads V1-V3 higher on the intercostal spaces than usual. The FX-8400 is able to detect Brugada syndrome without requiring moving these leads to a higher position. Rather, detection of Brugada syndrome is enhanced through the use of additional synthesized leads.

#### Synthesized High Intercostal Leads

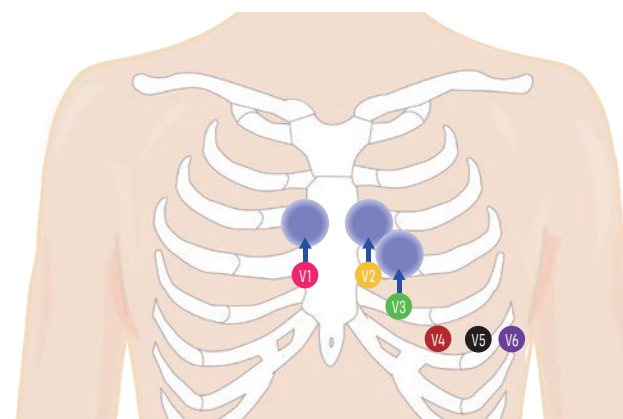

The synthesized electrocardiogram provides enhanced accuracy for detecting Brugada syndrome.



#### Improved Sensitivity to Brugada through an Automated Analysis

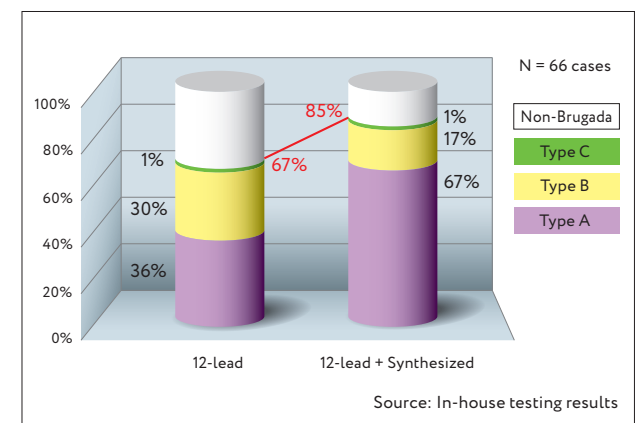

## Newest Interpretation Program

### TG Version

#### Comparison with Previous Version of Analysis Program

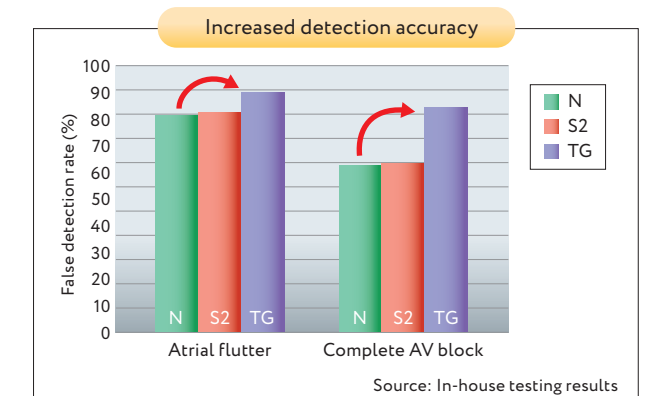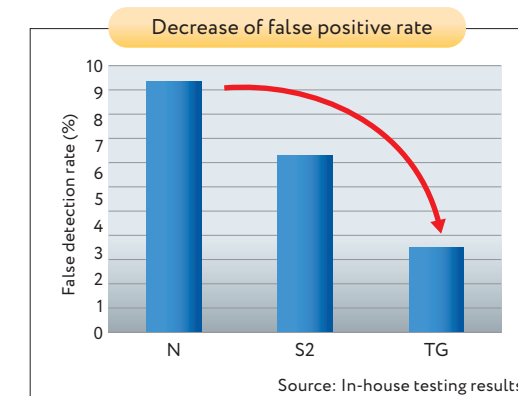


The TG version extracts P and f waves from the continuous waveforms, performs frequency analysis of extracted waveforms, and analyses the complete atrioventricular block and atrial fibrillation in addition to conventional P wave and f wave measurements.

# Added Support for Electrocardiograph Operators

## FX-8400 Will Appeal to Anyone Who Operates an Electrocardiograph System


### Auto Capture Function

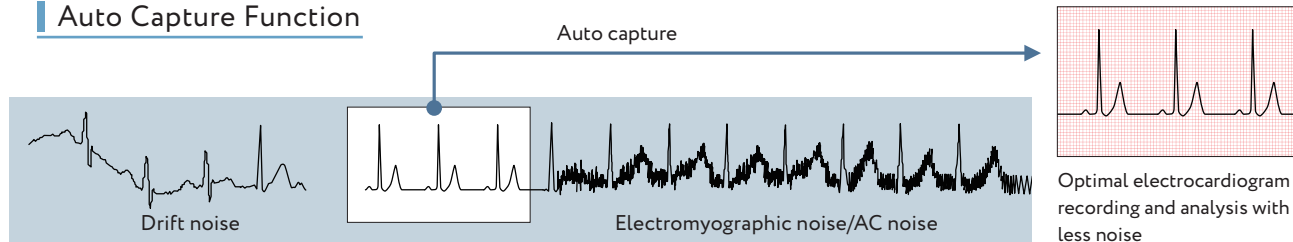

The Auto-Capture function will automatically select and save waveforms that have relatively less noise in order to facilitate improved interpretation and diagnosis.


### R Wave Detection Lead Auto Switch Function

If the lead being used to measure heart rate is producing an amplitude that is too small, the cardiograph will automatically select a more appropriate lead for this measurement.

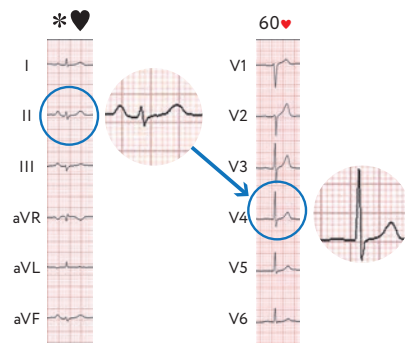


### Freeze Function

The waveform set obtained for an examination is stored in full for up to five minutes, giving time for the user to select the best portion to be used in the analysis.

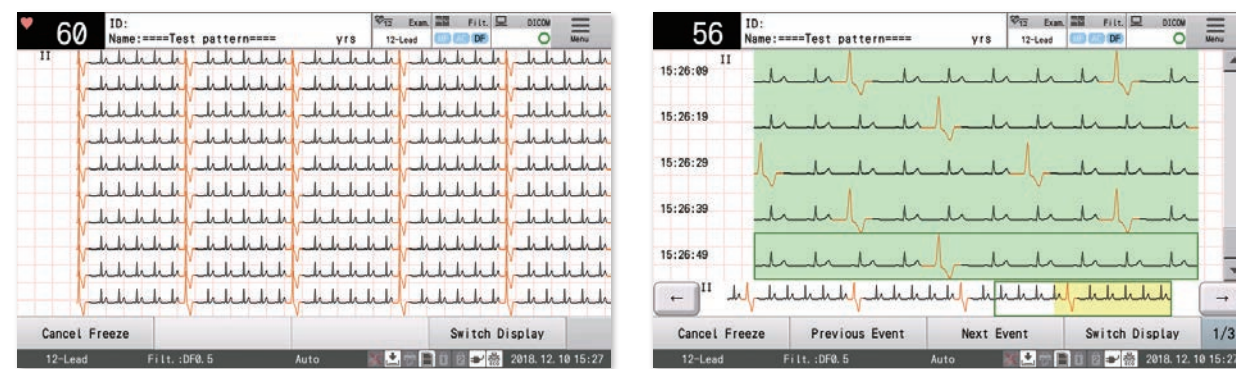

### Error Prevention Function


If the right or left electrode has been misplaced, the device will notify the user to check them.

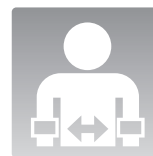

### Analysis Report



### Commentary Report

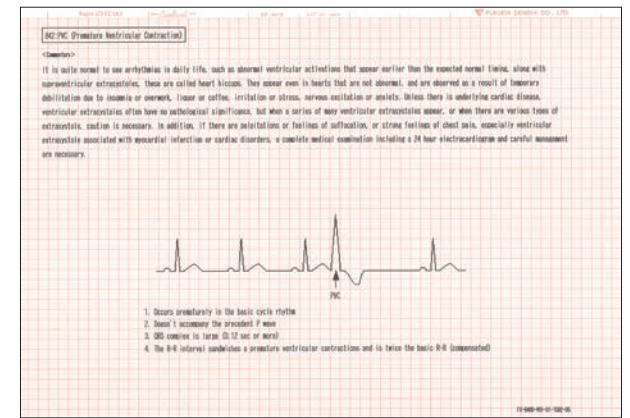

For the interpretation obtained during analyses, two types of reports are printed: the Analysis Report, which explains why the interpretation was given, and the Commentary Report, which provides a detailed explanation of that interpretation.

## Data Transfer Network

